ByFlo Products

Instruction Manual

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Arm Blood Pressure Monitor

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Model #: AOJ-30F

Contents

1	Before Using the Unit	2
2	Operating Instruction	5
2.1	Introduction of machine	5
2.2	Battery Installation / Removal	7
2.3	Settings	8
2.4	Proper Use of the Arm Cuff	10
2.5	Sitting Correctly	13
2.6	Taking a Measurement	13
2.7	Using the Memory function	15
2.8	Other Functions	16
3	Important Instructions	17
4	Common Q & A on Blood Pressure	21
5	Error Messages and Handling	23
5.1	Troubleshooting	24
6	Care and Maintenance	26
6.1	Cleaning, Disinfecting, and Disposal	26
6.2	Upkeep and Maintenance	28
7	Technical Information	29
7.1	Specifications	29
7.2	Technical description	31
8	Warranty Information	35

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1 BEFORE USING THE UNIT

Thank you for purchasing the Arm Blood Pressure Monitor. The monitor uses the oscillometric method of blood pressure measurement. This means the monitor detects your blood movement through your brachial artery and converts the movements into a digital reading.

The device can be used in homecare environment, and the patient is an intended operator, and all the functions can be safely used

Before use please open the package carefully and check whether all the parts are available according to the following packing list and whether the parts are damaged during Transportation and then instal, and operate in strict accordance with the manual.

The unit comes with the following components:

- ▶ Main Unit
- ▶ Arm Cuff
- ▶ Instruction Manual
- ▶ 4xAAA Batteries

SYMBOLS USED IN THIS INSTRUCTION MANUAL					
	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.				
	Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.				
	Please read this instruction manual thoroughly before using the unit. Please keep for future reference. For specific information about your own blood pressure, CONSULT YOUR DOCTOR.				
	Transport package shall be kept always from rain.		Keep away from sunlight		Fragile, handle with care
	AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY				
	Refer to instruction manual/booklet		Unique device identifier		
	MANUFACTURER		Date of manufacture		Made in China
	SERIAL NUMBER		Model number		Medical device
	Type BF applied part		Importer		CE mark
	The marking of electrical and electronics devices according to Directive 2002/96/EC. The device accessories and the packaging have to be disposed of waste correctly at the end of the usage. Please follow Local Ordinances or Regulations for disposal.				
	Device used within the Magnetic Resonance (MR) environment is prohibited.				

Intended purpose: The Arm Blood Pressure Monitor is intended to measure the systolic pressure and diastolic pressure, as well as the pulse rate of adult person via non-invasive oscillometric technique at medical facilities or at home.

Indication: Intermittent measurement of human body blood pressure from the upper arm.

Contraindications:

- ▶ The product can not be used for patients with arrhythmia.
- ▶ This product can not be used for infants, neonates or pregnancy.
- ▶ Individuals unable to express themselves should not use this product.

Intended users: Professional medical staff, lay persons.


Patient Population: Adults.

Clinical benefits: Measuring body blood pressure helps diagnose human body conditions.


2 OPERATING INSTRUCTION

2.1 INTRODUCTION OF MACHINE

2.1.1 Body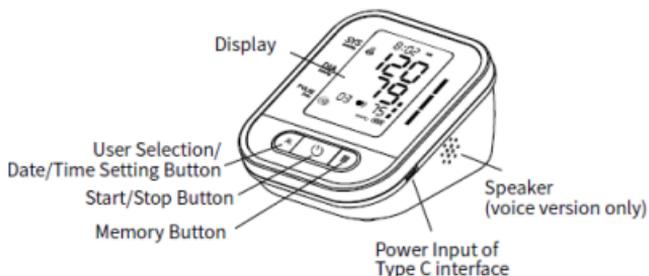


2.1.2 Display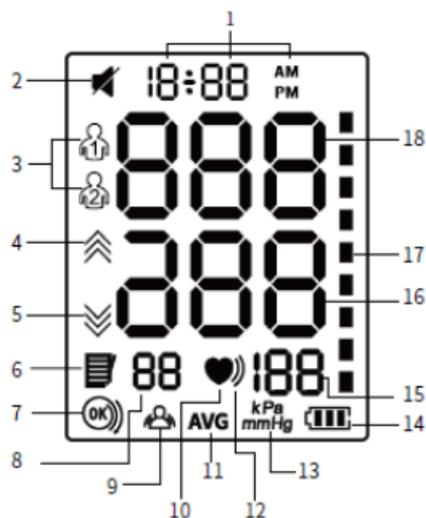

1. Date and Time
2. Voice on/off icon (voice version only)
3. User Icon
4. Inflation Icon
5. Deflation Icon
6. Memory Icon
7. Detection for Cuff Wrapping Icon
8. Memory Number
9. Indication for Wrong Operation
10. Heartbeat Icon
11. Average Value
12. Irregular Heartbeat Icon
13. Unit of Pressure
14. Low Battery
15. Pulse Rate
16. Diastolic Blood Pressure
17. Blood Pressure Classification
18. Systolic Blood Pressure

2.1.3 Arm Cuff

- Fit for 8.7-16.5 inches (22 cm-42cm) range of upper arm perimeter.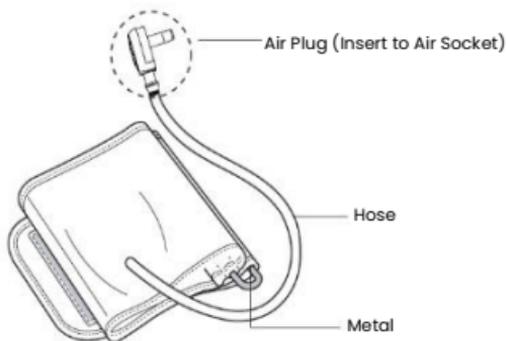


⚠ If your arm cuff is broken or not functional, please use a new cuff. Note that a new arm cuff does not come with an air plug. Kindly continue using the old air plug with the new arm cuff.

2.1.4 Power

4 AAA Alkaline Batteries

⚠ Do not use rechargeable batteries.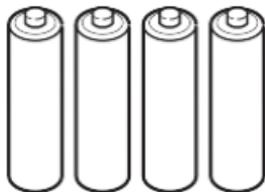


2.1.5 Instruction Manual

- Please keep the instruction manual well after using.

2.2 BATTERY INSTALLATION / REMOVAL

1. Open the battery cover according to the method shown in the figure.
2. Place 4 AAA batteries in the battery compartment and pay attention to the electrode indication of the batteries. Install the batteries as indicated in the picture.
3. Replace the battery cover.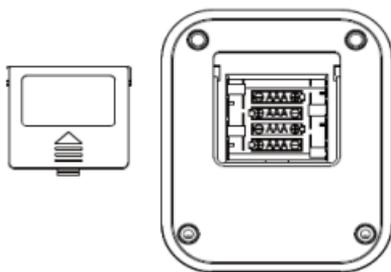

⚠ **Caution!**

- ▶ Remove the batteries if you do not intend to use the device for a long time (over 3 months).


2.2.1 Type-C connection for power supply

The cable is not included in the packing list. In addition to the batteries, the power can also be supplied by plugging into d.c. 5V external power supply through a Type-C port.

2.3 SETTINGS


2.3.1 User mode

In power-off mode, press the " 👤 " button to enter the user group selection interface. Then press the " 👤 " button again to switch and select user groups.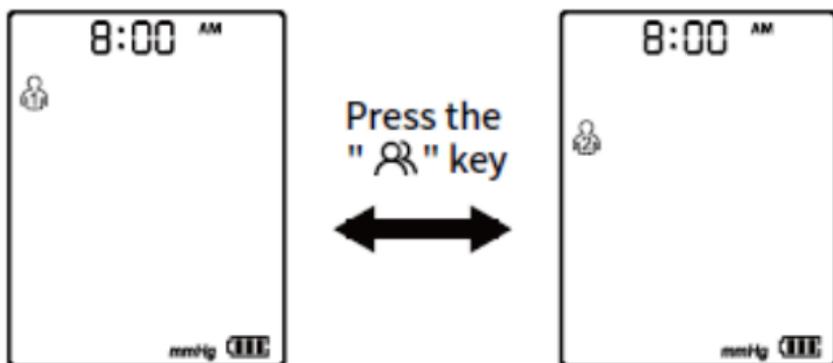


2.3.2 Date and time setting

In standby mode, press the " 👤 " button for about 3 seconds to enter the date setting, and "year" will flash. Press the " 👤 " button to adjust to the desired year, and then press the " 📅 " button to confirm the selection. When the "year" is set, it will automatically enter the month setting. At this time, the "month" icon will flash. You can switch to the desired value by pressing the " 📅 " button. Follow the same step to set "date", "hour", and "minute".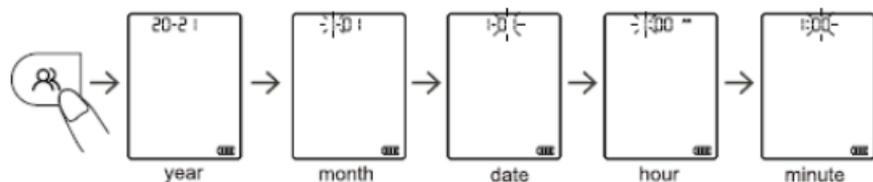

2.3.3 Unit display setting

In standby mode, keep pressing "⏻" button for about 10 seconds to enter the unit selection, and press "☰" to switch mmHg/kPa. Press "👤" to confirm the selection; The default unit is mmHg.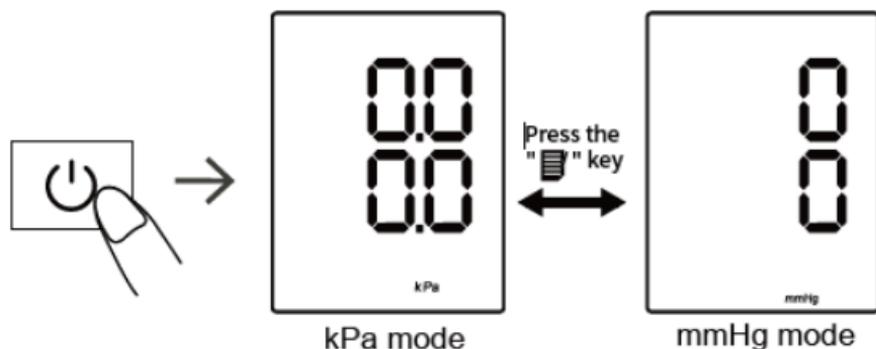

2.4 PROPER USE OF THE ARM CUFF


2.4.1 Preparation before measurement

- ▶ Remove the clothing on the arm.
- ▶ Always measure in the same arm (generally the left arm).
- ▶ Remain still and keep quiet during measurement.
- ▶ Relax as much as possible and do not talk during measurement.
- ▶ Measure your blood pressure at about the same time every day.
- ▶ Do not measure blood pressure immediately after physical exercise or a bath. Rest for 20 to 30 minutes before taking the measurement.
- ▶ Readings under the conditions listed below may affect results:


Within an hour after dinner, after having wine, coffee, tea, sports, talking, being nervous, being in unsteady mood, bending forward, moving, room temperature dramatically changing during measuring; inside a moving vehicle, repeated and continuous measuring.

2.4.2 Procedure


1. Connect the arm cuff to your monitor by inserting the air plug into the air jack securely.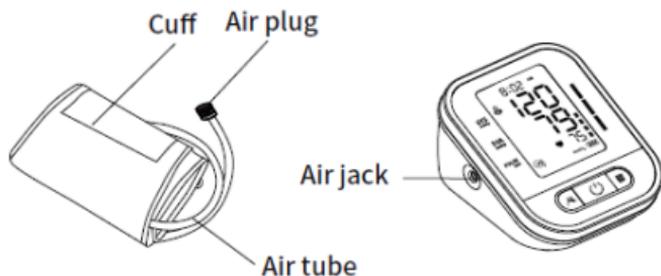

2. Place your hand through the cuff loop. Pull the cuff until it reaches your upper left arm.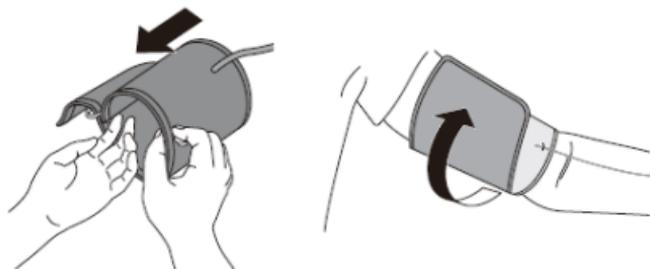

Note: The bottom edge of the arm cuff should be 0.79–1.18 inches (2 to 3 cm) above the inside elbow. The air tube is on the inside of your arm and aligned with your middle finger.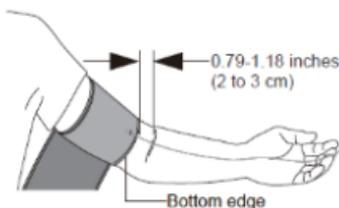

3. Make sure that the air tube is positioned on the inside of your arm and wrap the cuff securely, so it can not move around your arm.
4. When you take a measurement on your arm, the air tube should run along the side of your elbow, along the bottom of your arm. Be careful not to rest your arm on the air tube.

Note: Repeated measurement will result in blood congestion in the arm, which will affect the measurement result. How to improve: raise the left hand and hold the fist several times, or take off the cuff and rest for at least 2-3 minutes before taking the measurement. You can raise the left hand and hold the fist several times, or take off the cuff and rest for at least 2-3 minutes before taking the measurement.

2.5 SITTING CORRECTLY


1. To take a measurement you need to be relaxed and comfortably seated in a room with a comfortable temperature. Place your arm on the table.
2. Sit in a comfortable chair with your back and arm supported.
3. Keep your feet flat and your legs uncrossed.
4. The arm cuff should be placed on your arm at the same level as your heart, with the arm resting comfortably on a table.

2.6 TAKING A MEASUREMENT

1. Start measurement after wearing the cuff.
2. Press the "⏻" button and the monitor will begin to inflate. Please do not move or talk during the measurement.

All display symbols appear on the screen. The cuff starts to inflate automatically.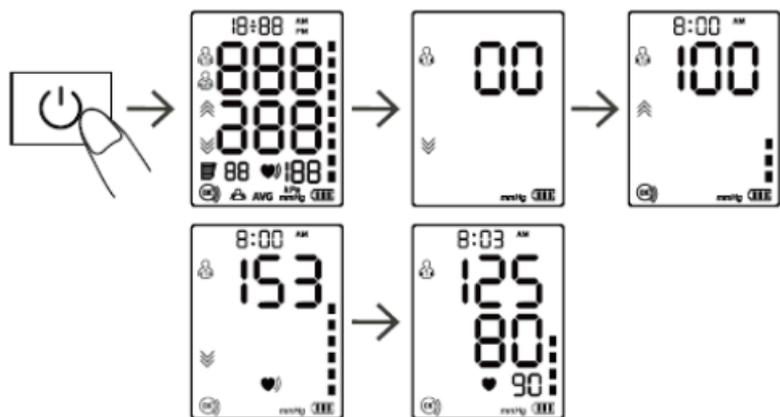

Note: If you feel uncomfortable during the measurement press the " ⏻ " button immediately to stop measure. When the air pressure is filled to a certain value, the value on the display screen will slowly drop at a certain speed and the heartbeat symbol will flash. After the measurement is completed, the systolic pressure diastolic pressure and pulse measurements will be displayed on the screen.

Note: Consult your doctor if unexpected readings are obtained.

Note: Please use the original cuff that has passed the clinical trial!

Any replacement of the original component with the component not provided by the manufacturer might result in measurement error.

2.7 USING THE MEMORY FUNCTION

This monitor stores 99 sets of memories of two users. After each measurement, the monitor will automatically store the measurement data. When the memory is full, the old measurement data will be overfilled with the new one.

2.7.1 To Enter Memory Mode

In standby mode, press the "☰" button once and the device will display the average value of the blood pressure measured for the first three times. Press the "☰" button again, the first memory will be displayed. Press the "☰" button again and the rest memories will be displayed one by

2.7.2 To delete memory values

Keep pressing the "☰" button for about 3 seconds to delete the memories of the selected user and the "∞" icon will appear on the screen.

2.8 OTHER FUNCTIONS

2.8.1 Self-check for cuff strap

The "OK" icon is always displayed on the screen when the cuff is worn correctly. When the cuff is too loose, the "OK" icon will always flash to remind you. If the "OK" icon is flashing, please press the "Power" button to stop the measurement.

2.8.2 "Keep Still" Indication

The "Keep Still" icon flashes when you move your body or shake your arm during the measurement, which may cause incorrect measurement results. Please adjust your posture and measure again.

3 IMPORTANT INSTRUCTIONS

- ▶ No maintenance or servicing when using.
- ▶ Do not operate when charging.
- ▶ Maintenance should be done by the manufacturer as suggested.
- ▶ Portable RF-communications equipment (including peripherals like antenna cables and external antennas) should be kept at least 30cm (12 inches) from the device and its parts to prevent degradation of the monitor.
- ▶ When the ambient temperature is less than 5°C, please take the device to the place where the ambient temperature is between 5°C~40°C at least 1 hour; When the ambient temperature is higher than 40°C, please take the device to the place where the ambient temperature is between 5°C~40°C at least 2 hours.
- ▶ DO NOT use this monitor for infants, toddlers, children or persons who cannot express themselves.
- ▶ DO NOT take medicine based on readings from the device. Contact your physician for specific information about your blood pressure. The patient should not self-diagnose or self-medicate per

measured results. Kindly adhere to the instructions of your physician or health provider.

- ▶ DO NOT use this monitor on an injured arm or an arm under medical treatment.
- ▶ DO NOT use the device while you are on an intravenous drip or blood transfusion.
- ▶ DO NOT use this monitor in areas containing high frequency (HF) surgical equipment, magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners. This may result in incorrect operation of the monitor and/or cause an inaccurate reading.
- ▶ Consult with your physician before using this monitor if you have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, arterial sclerosis, poor perfusion, diabetes, pregnancy, pre-eclampsia or renal disease.
- ▶ NEVER diagnose or treat yourself based on the readings. ALWAYS consult with your physician.
- ▶ To avoid strangulation, please keep the air tube and type-C charging cable away from the infants, toddlers and children.
- ▶ Stop using this monitor and consult with your physician if you experience skin irritation or discomfort.

- ▶ Consult with your physician before using this monitor if you have had a mastectomy.
- ▶ Consult with your physician before using this monitor if you have severe blood flow problems or blood disorders, because the cuff inflation can cause bruising.
- ▶ DO NOT use this monitor for any purpose other than measuring blood pressure and pulse rate.
- ▶ DO NOT disassemble or attempt to repair this monitor or other components. This may cause an inaccurate reading.
- ▶ DO NOT use in a location where there is moisture or a risk of water splashing this monitor. This may damage this monitor.
- ▶ DO NOT use this monitor in a moving vehicle such as in a car.
- ▶ DO NOT drop or subject this monitor to strong shocks or vibrations.
- ▶ DO NOT use this monitor in places with high/low humidity and temperatures.
- ▶ Do not use this device within regions of active HF-surgical equipment or RF-shielded room of an ME system for magnetic resonance imaging, where EM disturbances may be high.

- ▶ Do not use this device adjacent to or stacked on other equipment to prevent improper functioning. If such use is necessary, all accessories involved must be checked for normal operation.
- ▶ The use of accessories and transducers other than those specified or provided by the manufacturer is prohibited. Failure to adhere to this might result in increased electro- magnetic emissions or decreased electromagnetic immunity of the device.

4 COMMON Q & A ON BLOOD PRESSURE

Q1: Why is the blood pressure value obtained at home lower than that obtained at the hospital?

The blood pressure difference between home and hospital measurements is about 20 mmHg - 30 mmHg (2.7 kPa - 4.0 kPa). This is because individuals tend to be more relaxed at home than at the hospital.

In addition, when the device is placed at a position over the heart, the blood pressure value tends to be much lower than it actually is. Ensure the device is positioned right at the heart level.

Q2: Why is the blood pressure value obtained at home higher than that obtained at the hospital?

- ▶ The anti-hypertensive drug might have lost its efficacy. Kindly adhere to your doctor's instructions.
- ▶ The cuff might not be in the correct position. If the cuff is not placed right, no arterial pressure value will be obtained, and the blood pressure value might be much higher than it is. Therefore, properly position the cuff.
- ▶ The cuff is not tight enough. If the cuff is loose, the compression force might fail to transmit to the

artery, causing the blood pressure value to be much higher than it is. Therefore, re-adjust and tighten the cuff further.

- ▶ The patient is not sitting correctly during the measurement. Slouching, tilting, bending, and sitting cross-legged are not encouraged while taking blood pressure measurements due to increased abdominal pressure or the arm position being below the heart. Kindly take readings in the correct posture.

Q3: When can I obtain better measurements?

Measurements are best taken in the mornings right after you urinate or when your mind and body are stable. We recommend taking readings at the same time of the day, every time.

5 ERROR MESSAGES AND HANDLING

If the measurement is abnormal, any of the following symbols may appear. Kindly use the recommended method for measurement.

Errors	Cause/Solution
Er U	The pressure cannot reach 30 mmHg (4 kPa) in 12 seconds.
Er H	The inflation reaches 295mmHg, and it deflates automatically after 20ms.
Er 1	The pulse rate is not detected correctly.
Er 2	Too much disturbance (Move, talk, or magnetic disturbance during a measurement).
Er 3	The measurement result is abnormal.
Er 23	SYS value is lower than 57mmHg.
Er 24	SYS value is higher than 255mmHg.
Er 25	DIA value is lower than 25mmHg.
Er 26	DIA value is higher than 195mmHg.

5.1 TROUBLESHOOTING

Anomaly	Possible Fault	Solution
Failure to power on	Whether the power is insufficient	Replace the batteries or insert the Type-C charging cable for power supply
	Whether the positive and negative poles of the battery are installed reversely	Install the batteries correctly
No pressurizing	Whether the air tube plug is inserted tightly	Insert the air tube plug firmly into the jack
	Whether the air tube is broken or leaked	Please contact the dealer to replace with a new cuff

Unable to measure due to the display error	Whether the arm is moved when pressurization	Keep your arm and body still
	Whether you talk during the measurement	Keep quiet while measuring the blood pressure
Air leakage of the cuff	Whether the cuff is twined too loose	Please tighten the cuff
	The airbag of the cuff is ripped	Please contact the dealer to replace with a new cuff
 If the blood pressure still cannot be measured after trying the above-stated solutions, please contact the dealer. Do NOT attempt to disassemble the device by yourself.		

6 CARE AND MAINTENANCE

6.1 CLEANING, DISINFECTING, AND DISPOSAL

6.1.1 Cleaning

1. Remove the batteries before cleaning.
2. The body of monitor can be cleaned with a clean, soft cloth.
3. The body can be wiped with a clean soft cloth stained with a small amount of neutral detergent or water.

Do not use corrosive cleaning agents, and take care not to immerse any part of the monitor in any fluid.

6.1.2 Disinfection

Recommended Disinfecting Agent

75% medical alcohol

Steps:

1. Carefully wipe the device with a soft, clean cloth dampened with a small amount of the above disinfectant, and dry immediately with a soft, clean, dry cloth.

2. The body of the device can also be cleaned with a soft, clean cloth dampened with a small amount of 75% medical alcohol for disinfection.

Do not disinfect through methods like high-temperature steam or ultraviolet radiation. These might damage the device and reduce its service life.

It is suggested to disinfect the monitor before and after use each time. Each time of disinfection shall be completed within 1min. The number of repeated disinfection each time shall not exceed 2 times.

6.1.3 Disposal

Dispose of the monitor, other components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.

Notes

- Do not bend or crease the air tube excessively.
- Do not store the monitor or its components:
 - if the monitor or its parts is wet.
 - in locations with extreme temperatures, humidity, direct sunlight, dust, or corrosive gases.
 - in areas with a high risk of vibrations or shocks.

6.2 UPKEEP AND MAINTENANCE

water or neutral detergent

- ▶ Always keep the surface of the device clean and tidy, helpful to prolong its service life.
- ▶ If the device is dirty, please wipe with a dry soft cloth. If the dirt cannot be eliminated easily, wipe with a soft cloth stained with water or neutral detergent, and then dry with a dry cloth.
- ▶ No maintenance or maintenance required when operating the device.

Do not allow water or other liquids to flow into the device.

7 TECHNICAL INFORMATION

7.1 SPECIFICATIONS

Product name	Arm Blood Pressure Monitor	
Model	AOJ-30F	
Display	LCD screen	
Measuring Method	Oscillometric measurement	
Measuring Part	Upper arm	
Measuring range	Blood pressure value	SYS: 57~255 mmHg (7.6~33.4 kPa); DIA: 25~195 mmHg (3.33~26 kPa);
	Pulse rate	40~199 bpm
Accuracy	Blood pressure value	±3 mmHg (±0.4 kPa)
	Pulse rate	±5%
Low Battery	4.2V±0.1V: low battery; <4.0V±0.1V: item will be turned off	
Auto power-off	1 minute without operation	
Power Source	4xAAA d.c. 6V or d.c. 5V type C cable	
Applied part	Type BF	
Operation mode	Continuous operation	
IP classification	IP21	
Weight	About 230g (without batteries)	

Dimensions	118mm (l) x 98mm (w) x 61mm (h) (4.65 inches x 3.86 inches x 2.40 inches)		
Screen size	44.5mm (l) x 58.5mm (w) 3.2 inches		
Cuff size	22~42 cm (8.66~16.53 inches)		
Service life	5 years		
Protection against electric shock	Internally powered supply Class II ME equipment (AC adapter)		
Operating Environment	Temperature condition	5°C~40°C	If stored or used beyond the designated temperature and humidity range, it will not be used properly
	Humidity condition	15%~95%RH	
	Atmospheric condition	70kPa~106kPa	
Transportation and Storage Environment	Avoid strong impact, direct impact, exposure or rain during transportation. The packaged Blood Pressure Monitor shall be stored indoors at the temperature of -20°C~55°C and the relative humidity of 10%~93%, atmospheric Condition: 70kPa~106kPa. without corrosive gas and with good ventilation.		

7.2 TECHNICAL DESCRIPTION

1. All necessary instructions for maintaining BASIC SAFETY and ESSENTIAL PERFORMANCE with regard to electromagnetic disturbances for the excepted service life.
2. Guidance and manufacturer's declaration - electromagnetic emissions and Immunity.

Table 1 - Guidance and manufacturer's declaration - electromagnetic emissions

Emission test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	N.A.
Voltage fluctuations/flicker emissions IEC 61000-3-3	N.A.

Table 2 – Guidance and manufacturer’s declaration – electromagnetic Immunity

Immunity Test	IEC 60601-1-2 Test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	Not applicable	Not applicable
Surge IEC 61000-4-5	Not applicable	Not applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Not applicable	Not applicable
Power frequency magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC61000-4-6	Not applicable	Not applicable
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz
NOTE UT is the a.c. mains voltage prior to application of the test level.		

Table 3 – Guidance and manufacturer’s declaration – electromagnetic Immunity

	Test Frequency (Mhz)	Band (Mhz)	Service	Modulation	Maximum Power (w)	Distance (m)	IEC 6060-1-2 Test level (v/m)	Compliance level (v/m)
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF Wireless communications equipment	385	380-390	TETRA 400	Pulse modulation 18Hz	1,8	0,3	27	27
	450	430-470	GMRS 460, FRS 460	FM ±5 kHz deviation 1kHz sine	2	0,3	28	28
	710	704-787	LTE Band 13,17	Pulse modulation 217Hz	0,2	0,3	9	9
	745							
	780							
	810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18Hz	2	0,3	28	28
	870							
	930							
	1720	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1,3,4,25; UMTS	Pulse modulation 217 Hz	2	0,3	28	28
	1845							
1970								
2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n,	Pulse modulation 217 Hz	2	0,3	28	28	

			RFID 2450, LTE Band 7					
	5240	5100- 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0,2	0,3	9	9
	5500							
	5785							

Table 4 - Guidance and manufacturer's declaration - electromagnetic Immunity

Test frequency	Modulation	IMMUNITY TEST LEVEL (A/m)
30 kHz	CW	8
134,2 kHz	Pulse modulation ^a 2,1 Khz	65
13,56 MHz	Pulse modulation ^a 50 Khz	7,5

NOTE:

- ▶ Optional AC adaptor should comply with the requirement of IEC 60601-1 standard.
- ▶ Use only the exclusive AC adapter specified by authorized dealers. Other AC adapter may vary in output voltage and polarities and may represent a risk on your life and damaging the device.

8 WARRANTY INFORMATION

- ▶ The unit is guaranteed to be free of defects in workmanship and materials when used as intended for a period of two years from the date listed on the purchase record.
- ▶ For repairs under this warranty, our authorized service agent must be notified of the fault within the warranty period. This warranty covers parts and labor only under normal operations. Any defects resulting from natural causes, such as floods or hurricanes, are not covered by this guarantee.
- ▶ This guarantee does not cover damage incurred by not following the instructions, accidental damage, or tampering/servicing by unauthorized service agents. Any transportation or freight fees incurred will be the owner's responsibility.
- ▶ This guarantee specifically excludes expendables and consumables, for example batteries.

All warranty claims must be directed to the distributor responsible for the sale of the device.

The content of this warranty is subject to change without further notice.

- ▶ Monitor subjected to misuse, abuse and neglect of these manual content excluded from the warranty.
- ▶ **WARNING:** No modification of this equipment is allowed.

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ByFlo Products Inc.