ByFloProducts™

Forehead & Ear Infrared Thermometer

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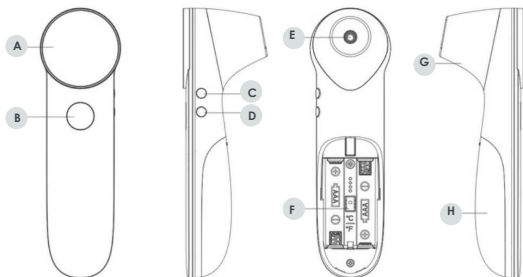
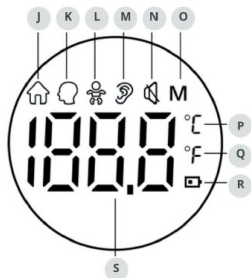
G
Probe cover**

H
Battery cover

*Child Mode: Recommended for people under 12 years of age

**Automatically switch from object, forehead and ear modes:

- Probe cover On: Forehead and Object mode
- Probe cover Off: Ear mode

J
Object temperature mode

K
Forehead temperature mode

L
Child mode

M
Ear temperature mode

N
Mute / Unmute

O
Memory

P Q
Temperature units (°C/°F)

R
Low battery alert

S
Temperature value

SAFETY PRECAUTIONS

Read the following precautions carefully before using this thermometer

ATTENTION

- Take care of the temperature probe lens, which is fragile.
- No service /maintenance while the equipment is in use.
- Dispose used batteries with care. To protect the environment, you are recommended to send the used batteries to a designated collection point.
- The thermometer doesn't need recalibration.
- Make sure that the thermometer is not exposed to lint, dust, light (including sunlight), etc.
- Please note the effects of degraded sensors that can degrade performance or cause other problems.
- Make sure that the thermometer is not exposed to pets, pests.
- If the thermometer is soiled or its infrared optical components is damaged, please stop using it.
- The lay operator or lay responsible organization should contact the manufacturer or manufacturer's representative on the following issues: -assistance in setting up, using, or maintaining the equipment or system when needed, or to report unexpected operation or events.
The IR thermometer is identified as intended for consumer use.
- The patient is an intended operator.
- The effect that the following actions could be unsafe as applicable: use of accessories, detachable parts, and materials not described in the instructions for use or modification of the equipment.
- The hazards that can result from unauthorized modification of the equipment.
- The manufacture can provide the circuit diagram, component part list, description and calibration instructions to assist service personal for parts repair.
- Do not subject the thermometer to vibration or impact.
- Do not use the thermometer on newborns or for continuous temperature monitoring purposes.
- Do not use the thermometer for purposes that are not specified in this User Manual. Follow the instructions in the "Measurement Process" chapter and carefully operate the thermometer when measuring children's temperature.
- Do not immerse the thermometer in water or other liquid, as it is not waterproof. Clean and disinfect the thermometer as described in the "Cleaning and Disinfection" chapter.
- Do not touch the tip of the temperature probe, on which a precise temperature sensor resides.
- Keep the temperature probe clean to make sure accurate readings.
- Before measuring the temperature from the ear canal, clean the earwax, if any.
- The ambient temperature must not be extremely high or low. To make sure accurate readings, keep the thermometer under room temperature for more than 30 minutes before use.

- Do not use the thermometer under an ambient temperature higher than 40°C (104°F) or lower than 10°C (50°F), which is beyond the operating temperature range of the thermometer.
- Risk of pollution! The user is recommended to send the overdue thermometer to local garbage disposal site or send it back to us.
- 2 AAA batteries of 1.5V are the only replaceable accessories of the thermometer. Please do not use the batteries of other voltages or specifications.

**WARNING**

- Do not force the temperature probe of the thermometer into an ear canal. Otherwise, the ear canal may get injured.
- Keep the thermometer out of the reach of children.
- The result may be inaccurate if you use the overdue thermometer.
- The thermometer is not intended to diagnose or treat any health problem or disease. The measurement results are for reference only.
- It is dangerous to make a self-diagnosis or self-treatment based on the obtained measurement results. For such purposes, please consult a doctor.

- ⊘ Do not charge an alkaline dry-cell battery or throw it in fire. Otherwise, the battery may explode.
- Do not disassemble the thermometer or attempt to repair it. Otherwise, the thermometer may be damaged permanently.
- Do not take temperature measurements on body parts other than forehead and ears. Otherwise, the temperature readings may be inaccurate.
- During measurement, do not use a mobile phone or any other device that may cause electromagnetic interference.
- Do not use the thermometer in an environment where flammable anesthetic mixture with air or with oxygen or nitrous oxide is available.

ENGLISH

SYMBOLS

Symbol	Description
	Type BF applied part.
	Attention must be paid.
	The action is prohibited.
	Information about the manufacturer.
	Date of manufacture.
	Consult the instructions for use.
	Waste electrical materials should be sent to a dedicated collection point for recycling. Must contact its local authorities to determine the proper method of disposal of potentially bio hazardous parts and ACCESSORIES.
IP22	Degree of protection against the Ingress of water.
 Warning	A personal injury or damage to the thermometer may occur if the thermometer is not used correctly.
 Attention	Inaccurate reading or damage to the thermometer may occur if the thermometer is not used correctly.

CHART REFERENCE

BODY PART	NORMAL TEMPERATURE RANGE
Forehead	34.7°C-37.3°C / 94.5°F-99.1°F
Ear Canal	35.8°C-38.0°C / 96.4°F-100.4°F
Mouth	35.5°C-37.5°C / 95.9°F-99.5°F
Armpit	34.7°C-37.3°C / 94.5°F-99.1°F
Anus	36.6°C-38.0°C / 97.9°F-100.4°F

Please use this chart as reference only. For more precise measurements, speak to your doctor.

AUDIOVISUAL REFERENCE OF YOUR THERMOMETER

MODE	TEMPERATURE RANGE	SOUND	RESULTS
Forehead (Adult/Child)	22.0°C-37.5°C / 71.6°F-99.5°F	A long beep	NORMAL
	37.6°C-43°C / 99.6°F-109.4°F	3 short double beeps	HIGH Flashing Value
Ear Canal (Adult/Child)	34.0°C-37.5°C / 93.2°F-99.5°F	A long beep	NORMAL
	37.6°C-43°C / 99.6°F-109.4°F	3 short double beeps	HIGH Flashing Value
Object	0°C-100°C / 32.0°F-212°F	A long beep	-

NOTE: When the forehead and ear temperature is between 37.6°C/99.6°F and 43.0°C/109.4°F, there will be 3 short double beeps and the temperature value will flash on the screen. This indicates that your body temperature is a high. You may have a fever. Please consult your doctor if you are not sure.

----- ENGLISH

IMPORTANT INFORMATION

Please note that the temperature of the human body varies throughout the day. It can also be influenced by numerous external factors: age, sex, type and thickness of skin. The temperature should be taken in a neutral environment. The person must not have been undertaken vigorous physical activity prior to taking his/ her temperature and the room temperature must be moderate. Also note that sitting and resting temperature is lower by about 0.5 °F to 0.7 °F (0.3 °C to 0.4 °C) than when standing.

Beware to not make a self-diagnosis or self-treatment based on the obtained measurement results. For such purposes, please consult a doctor.

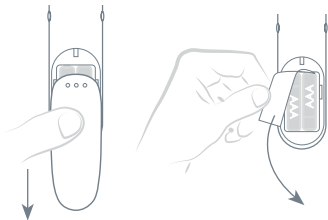
TO ENSURE RELIABLE AND STABLE RESULTS

- Do not hold the thermometer in your hands for too long because it is sensitive to the ambient temperature.
- Before the first use and each time there is a significant change in the ambient temperature, make sure to wait 10 to 15 minutes so the thermometer can acclimate to the new environment.
- The thermometer is meant to be used indoors without a strong air convection (e.g. from air conditioning, etc.) between the thermometer and the person/object.
- In order to take more than one measurement consecutively, please allow 3 to 5 seconds interval between two measurements.
- Push back hair from the forehead and wipe away any perspiration when measuring temporal temperature.
- Make sure that the ear canal is clean and dry before using.

BEFORE USE

INITIATING BATTERIES

When using the thermometer for the first time, slide open the battery cover **H** and move the battery's insulating plastic away.

SOUND CONTROL

To mute: Turn on the thermometer **B** and immediately press and hold 3 seconds on the sound button **C**. The mute symbol will appear on the screen .

To unmute: Turn on the thermometer **B** and immediately press and hold 3 seconds on the sound button **C**. The thermometer will beep once and the mute symbol will disappear from the screen.

SWITCHING FROM METRIC READING (°F/°C)

Remove the battery cover **H** and switch the metric unit by moving the "°C/°F" toggle **F** up or down. The symbol will change on the screen.

RETRIEVE READINGS

1. Turn on the thermometer **B** and press the Memory button **C**. "01" will appear on your screen for the most recent reading. After 2 seconds, the latest reading will appear.
2. To view older readings, press on the memory button again.
3. When the maximum number of records is exceeded (20 readings), the oldest memory data will be overwritten.

NOTE: The readings are saved from most recent to oldest, being 01 the most recent saved reading.

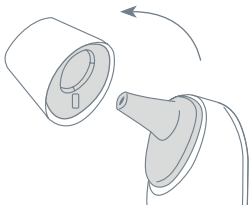
CLEAR MEMORY

To clear all the memory data, Turn on the thermometer **B**, slide open the battery cover **H** and remove the batteries completely. Wait 10 seconds and re-install the batteries. The memory data will be then reset.

DIFFERENT MEASURING METHODS

MEASURING EAR TEMPERATURE

1. Take the probe cover **G** off from the thermometer before measuring the ear temperature.
2. Press the Power button **B** to turn on the thermometer. It should already be set on ear mode and the ear symbol  should appear on the screen.
3. The device is automatically set on *adult* mode. To measure the temperature of a child, set device on *child* mode by pressing on the mode button **D**. The baby icon  will appear next to the ear symbol  on the screen.
4. Insert the temperature probe into the ear canal.


FOR CHILDREN FROM 3 MONTHS TO 1 YEAR

When taking the temperature of a child, gently pull the ear backwards laterally to open the ear canal properly, so that the temperature probe can send an infrared ray from the eardrum.

 Do not force the thermometer into the ear canal. Otherwise, the ear canal may get injured.

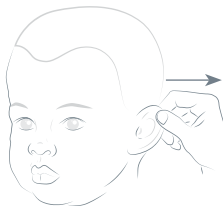 Be careful when taking temperature on a child, whose ear canal is small.

FOR CHILDREN AGED 1 YEAR TO ADULT

When taking the temperature on an adult, gently pull the ear up and back to make sure the ear canal is straight, so that the temperature probe can receive an infrared ray from the eardrum.

5. Press and release the Measure button **B**. The ear temperature reading will appear on the screen instantly.
6. If no activity is detected, the thermometer will power off automatically in 10 seconds.

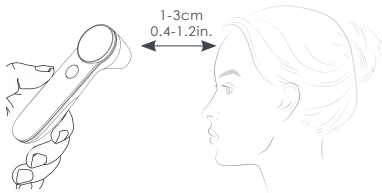
NOTE:

- (1) After each measurement, clean the probe with a soft cloth, and put the thermometer in a dry and well-ventilated place. If the person has a fever, clean probe to disinfect the probe. Follow disinfecting instructions in this manual. Be careful to not rub the sensor.
- (2) Make sure that the ear canal is clean and dry before starting a measurement. It is recommended to clean the ear canal with a cotton swab if any dirt exists. Otherwise, the temperature probe may be polluted, and temperature readings may be inaccurate.
- (3) Do not use the ear mode temperature if the ear is infected with otitis or suppuration.
- (4) The ear mode is suggested for babies from 3 months through adulthood.

MEASURING FOREHEAD TEMPERATURE

1. Make sure the probe cover **G** of thermometer is on the device.
2. Press the Power button **B** to power on the thermometer.
3. When measuring the temperature of an adult, set device on *Forehead* mode by clicking on the Mode button **D**. The head icon  will appear on the screen. When measuring the temperature of a child, set device on *Child* mode. The baby icon  will appear on the screen next to the head icon .
4. Point the thermometer probe to the center of the forehead, about 1 to 3 cm (0.4 to 1.2 in) away from the skin surface.

5. Press and release the Measure button **B**. The temperature reading will be displayed on the screen instantly.
6. If no activity is detected, the thermometer will power off automatically in 10 seconds.

NOTE: The forehead mode is suggested to use on children as of the age of two up to adulthood.

MEASURING OBJECT TEMPERATURE

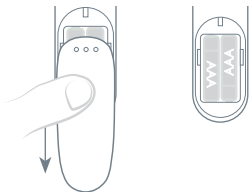
1. Make sure the probe cover **G** of thermometer is on the device.
2. Press the Power button **B** to power on the thermometer.

ENGLISH

3. Set the device on Object mode by clicking on the Mode button **D**. The house icon  will appear on the screen.
4. Point the thermometer probe to the center of the object, about 1 to 3 cm (0.4 to 1.2 in) away from the object surface.
5. Press and release the Measure button **B**. The temperature reading will be displayed on the screen instantly.
6. If no activity is detected, the thermometer will power off automatically in 10 seconds.

REPLACING BATTERIES

1. Slide the battery cover **H** off along the marked direction and take it off.
2. Insert the two AAA batteries into the compartment according to the stated polarities.

NOTE:

- (1) Make sure that the batteries are installed correctly. Otherwise, the thermometer might get damaged.
- (2) If the low-battery symbol appears on the screen, replace the batteries.
- (3) Make sure you replace the batteries with the same type of batteries.

CLEANING & MAINTENANCE

CLEANING YOUR THERMOMETER

1. Take the batteries out before cleaning.
2. Clean the probe **E** with a soft cloth. Clean the lens of the temperature probe with a cotton swab.
3. Wipe the thermometer body with a slightly damp soft cloth.

 Keep water out off the lens sensor during the cleaning process, otherwise you could damage the lens. The lens might get scratched if it is not cleaned properly, which might cause inaccurate readings.

 Do not clean the thermometer with corrosive cleansers. During the cleaning process, do not immerse any part of the thermometer into liquid, or allow any liquid to penetrate the thermometer.

Recommended cleaning product:

- Medical cleaners
- Mild soap

DISINFECTING YOUR THERMOMETER

1. Wet a clean soft cloth with a small quantity of disinfectant.
2. Wipe the thermometer and quickly dry it.
3. Disinfect the thermometer body and the area around the temperature probe with a cloth slightly moistened with 75% medicinal alcohol.

 Do not use hot steam or ultraviolet radiation for disinfection. Otherwise, the thermometer may be damaged or quickly aged.

 It is recommended to disinfect the thermometer before and after each use. The disinfection time is completed within 1 minute, and the number of repetitions per disinfection is not more than 2 times.

Recommended disinfectants:

- Isopropyl alcohol solution (70% concentration)
- Medicinal alcohol (75% concentration)
- Sodium hypochlorite solution (3% concentration)

MAINTENANCE

- Ensure the safety of the thermometer, and keep an eye on whether it has potential safety hazards in normal use each week (e.g. whether the lens is broken, the shell has cracks and the sensing head is polluted). Do not use the thermometer with potential safety hazard. Clean the thermometer if not used for a long time.
- After each use, clean the temperature probe as described in the previous **Cleaning and Disinfection** section.
- Store the thermometer in a dry, dust-free, and well-ventilated place. Make sure that the thermometer is not exposed to direct sunlight. Make sure that the storage and transportation environments meet the safety requirements.
- Check regularly whether safety risks exist.
- Remove the batteries if the thermometer will not be used for more than two months.

----- ENGLISH

ENVIRONMENT & RECYCLING

This appliance has been designed and manufactured with high-quality materials and components that can be recycled and reused.

Do not dispose of this product with ordinary household waste. Find out the proper procedure to dispose of electronics in your region. Take this product to a collection point for recycling electrical and electronic equipment.

WARNINGS

- Follow the maintenance advice found in this instruction manual to insure reliable stable results.
- The thermometer may be used for professional purposes or for personal use.
- The thermometer must be used for the purposes described in this instruction manual, in an ambient temperature ranging from 50 °F (10 °C) to 104 °F (40 °C).
- The thermometer must always be kept in a clean, dry area.
- Do not expose the thermometer to electric shocks or to extreme temperatures over 131 °F (55 °C) or under -4 °F (-20 °C).
- Do not use the thermometer when relative humidity is higher than 85% or lower than 15%.

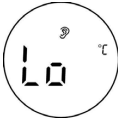
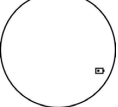
INTENDED USE

The dual mode Digital Infrared Thermometer takes the human body temperature via the eardrum or the forehead for both home and professional use. The forehead mode is indicated for use by people above two years old and the eardrum mode is indicated for use by people above three months old.

TROUBLESHOOTING

PROBLEM	POSSIBLE CAUSE	SOLUTION
The thermometer fails to power on.	Low battery	Change the batteries
	The polarities of the batteries are reversed.	Make sure that the batteries are installed correctly.
	The thermometer is damaged.	Contact the manufacturer.
"ER1" is displayed.	The ambient temperature is lower than 50 °F (10 °C) or higher than 104 °F (40 °C).	Take the measurement under an ambient temperature between 50 °F (10 °C) and 104 °F (40 °C).
The temperature reading is lower than the typical body temperature range.	The lens of the temperature probe is dirty.	Clean the lens using a cotton swab.
	The thermometer probe is not aligned to the eardrum.	Reposition the thermometer probe so that it is aligned to the eardrum.
	The thermometer is used within 30 minutes after being taken from a cold environment.	Wait for more than 30 minutes after the thermometer is moved into the measurement environment.
The temperature reading is higher than the typical body temperature range.	The temperature probe is damaged.	Contact the manufacturer.

TROUBLESHOOTING

PROBLEM	CAUSE	SOLUTION
	<p>In Ear mode, a temperature reading of less than 34.0°C(93.2.0°F). In Forehead mode, a temperature reading of less than 22.0°C(71.6°F). In Object mode, a temperature reading of less than 0°C(32.0°F).</p>	<p>Make sure to take the measurement within the temperature range listed in this manual.</p>
	<p>In Ear mode, a temperature reading of more than 43.0°C(109.4°F). In Forehead mode, a temperature reading of more than 43°C (109.4°F). In Object mode, a temperature reading of more than 100°C (212.0°F).</p>	<p>Make sure to take the measurement within the temperature range listed in this manual.</p>
	<p>The ambient temperature is higher than 40.0°C (104.0°F) or lower than 10.0 °C (50.0°F).</p>	<p>Take the measurement under an ambient temperature between 50 °F (10 °C) and 104 °F (40 °C).</p>
	<p>This error occurs when data is being read from or written to the memory, or the temperature correction is not complete.</p>	<p>Wait 10s, and retake the measurement.</p>
	<p>When the battery voltage is lower than 2.5V ± 0.1V, the low battery symbol will appear on the display.</p>	<p>Replace the batteries.</p>

TECHNICAL SPECIFICATIONS

THERMOMETER	
Product Name	Forehead & Ear Infrared Thermometer
Product Model	JPD-FR301
Power Supply Mode	Internal Power Supply
Operating Voltage	DC 3V
Battery Model	AAA x 2
Operating Mode	Continuous operating
Display	Segment LCD
Measure Time	About 1 second
Latency Time	About 3 seconds
Measuring Range	Forehead Mode: 22.0 °C - 43.0 °C (71.6 °F - 109.4 °F)
	Ear Mode: 34.0 °C - 43.0 °C (93.2 °F - 109.4 °F)
	Object Mode: 0 °C - 100 °C (32 °F - 212.0 °F)
Accuracy (Laboratory)	Forehead Mode: ± 0.2 °C (36 °C - 39 °C) ± 0.3 °C (22 °C - 36 °C / 39 °C - 43 °C)
	Ear Mode: ± 0.2 °C (36 °C - 39 °C) ± 0.3 °C (34 °C - 36 °C / 39 °C - 43 °C)
	Object Mode: ± 1 °C (± 2 °F)
Accuracy (Clinical)	± 0.3 °C (± 0.6 °F)
Child Mode	Recommended for people under 12 years of age
Memory	20 temperature readings
Low-Battery Alert	The low-battery symbol is displayed if the power voltage is lower than 2.4 V \pm 0.1V
Automatic Power-off	The thermometer automatically powers off if it is not used in 10 \pm 1 seconds.
Outer dimensions	145.5 x 43.4 x 45.4 mm
Weight	94 g (with batteries)
Operating Environment	Temperature: 10°C~ 40°C (50°F~104°F)
	Humidity: 15%~95% RH, non-condensing
	Atmospheric pressure: 86~106 kPa

WARRANTY AND AFTER-SALE SERVICE

The device is under warranty for one year from the date of purchase.

The batteries, the packaging, and any damage caused by improper use are not covered by the warranty.

Excluding the following user-caused failures:

1. Failure resulting from unauthorized disassembly and modification.
2. Failure resulting from an unexpected dropping during application or transportation.
3. Failure resulting from not following the instructions in the user's manual.

EMC INFORMATION-GUIDANCE AND MANUFACTURE'S DECLARATION

- ⚠ The Infrared Thermometer JPD-FR301 needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided for in the ACCOMPANYING DOCUMENTS.
- ⚠ Portable and mobile RF communications equipment can affect this Infrared Thermometer JPD-FR301.
- ⚠ The Infrared Thermometer JPD-FR301 should not be used adjacent to or stacked with other equipment.

Guidance and manufacturer's declaration – Electromagnetic emission –for all equipment and systems

The Infrared Thermometer JPD-FR301 is intended for use in the electromagnetic environment specified below. The customer or the user of the Infrared Thermometer JPD-FR301 should assure that it is used in such an environment.

GUIDANCE AND MANUFACTURER'S DECLARATION - ELECTROMAGNETIC EMISSIONS		
EMISSIONS TEST	COMPLIANCE	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
RF emissions CISPR 11	Group 1	The Infrared Thermometer JPD-FR301 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Infrared Thermometer JPD-FR301 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

GUIDANCE AND MANUFACTURER'S DECLARATION - ELECTROMAGNETIC IMMUNITY			
IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Electrostatic discharge (ESD) IEC 61000-4-2	±6kV contact ±8 kV air	±6kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Power frequency magnetic field IEC 61000-4-8	3 A/m 50Hz/60Hz	3 A/m 50Hz/60Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Radiated RF IEC 61000-4-3	3 V/m 80 MHz - 2.5 GHz	3 V/m	<p>Portable and mobile RF communications equipment should be used no closer to any part of the JPD-FR301, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance</p> $d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$ <p>80 MHz to 800 MHz</p> $d = \left[\frac{7}{E_1} \right] \sqrt{P}$ <p>800 MHz to 2.5 GHz</p> <p>where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an Electromagnetic site survey, a should be less than the compliance level in each frequency range. b Interference may occur in the vicinity of equipment marked with the following symbol: </p>
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.</p> <p>A. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the JPD-FR301 is used exceeds the applicable RF compliance level above, the JPD-FR301 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the JPD-FR301.</p> <p>B. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m</p>			

Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEM -for EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING

The Infrared Thermometer JPD-FR301 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Infrared Thermometer JPD-FR301 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Infrared Thermometer JPD-FR301 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (M)	
	80 MHz to 800 MHz $d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$
0.01	0.12	0.23
0.1	0.38	0.73
1	1.2	2.3
10	3.8	7.3
100	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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