size:248*375mm

DIGITAL THERMOMETER

INSTRUCTIONS MANUAL

Model: DMT-4132

WARNING

- Read instructions thoroughly before using digital thermometer.
- Choking Hazard: Thermometer cap and battery may be fatal if swallowed. Do not allow children to use this device without parental supervision.
- Do not use thermometer in ear. Designed use is for oral, rectal, and armpit (axillary) readings only.
- If the unit has been stored at temperatures over 5°C ~ 40°C (41°F ~ 104°F), leave it in 5°C ~ 40°C (41°F ~ 104°F) ambient temperature for about 15 minutes before using it.
- Remove battery from the device when not in operation for a long time.
- The use of temperature readings for self-diagnosis is dangerous. Consult your doctor for the interpretation of results. Self-diagnosis may lead to the worsening of existing disease conditions.
- Do not attempt measurements when the thermometer is wet as inaccurate readings may result.
- Do not bite the thermometer. Doing so may lead to breakage and/or injury.
- Do not attempt to disassemble or repair the thermometer. Doing so may result in inaccurate readings. After each use, disinfect the thermometer especially in case the device is used by more than one person. Do not force the thermometer into the rectum. Stop insertion and abort the measurement when pain is present. Failure to do so may lead to injury.
- Do not use thermometer orally after being used rectally.
- For children who are two years old or younger, please do not use the devices orally.
- If the unit has been stored at temperatures over 5°C ~ 40°C (41°F ~ 104°F), leave it in 5°C ~ 40°C (41°F ~ 104°F) ambient temperature for about 15 minutes before using it.
- ME equipment should not be cleaned and disinfected while in use.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

INTENDED USE

The digital thermometers are intended to measure the human body temperature in regular mode orally, rectally or under the arm. And the devices are reusable for clinical or home use on people of all ages, including children under 8 years old with adult supervision.

INTENDED USER

- The patient is an intended operator. The Applied part is the probe.
- All functions the patient can safely use.
- The patient can replace the batteries.

INDICATION FOR USE

The intended purpose of this device is for some patient who want to monitor their body temperature.


PLEASE READ CAREFULLY BEFORE USING

This digital thermometer provides a quick and highly accurate reading of an individual's body temperature. To better understand its functions and to provide years of dependable results, please read all instructions first. This appliance conforms to the following standards: ISO 9001-2-56 Medical electrical equipment -Part 2-56 Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement, EN 60601-1-11 Medical electrical equipment -Part 1-11: General requirements for basic safety and essential performance - Collateral Standard Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and complies with the requirements of EN 60801-1-2(EMC), IEC/EN 60601-(Safety) standards. And the manufacturer is ISO 13485 certified.

CONTENTS

1 Thermometer, 1 Instructions manual, 1 Storage Case

PRODUCT ILLUSTRATION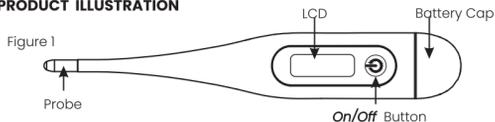

PRECAUTION

The performance of the device may be degraded should one or more of the following occur:
- Operation outside the manufacturer's stated temperature and humidity range.
- Storage outside the manufacturer's stated temperature and humidity range.
- Mechanical shock (for example, drop test) or degraded sensor.
Patient temperature is below ambient temperature.
Portable and mobile RF communications can affect the device. The device needs special precautions regarding EMC according to the EMC information provided in the accompany documents.

SYMBOL EXPLANATION

	Caution		TYPE BF APPLIED PART		Manufacturer
	Date of manufacture		Lot/ Batch Code		Refer to instruction manual
	Storage and Transportation Humidity limitation: 15%-95%RH		MD		General symbol for recyclable
	Atmospheric pressure limitation: 70kPa-106kPa		MD		Medical Device
	Disposal of this product and used batteries should be carried out in accordance with the national regulations for the disposal of electronic products.				
	Storage and Transportation Temperature Limit: -20°C-55°C(-4°F-131°F)				
	The first num:2: Protected against solid foreign objects of 12.5 mm diameter and greater. The second num:7: Protected against the effects of temporary immersion in water.				


SPECIFICATIONS

Type:	Digital Thermometer (Not Predictive)
Measure Range:	32.0°C ~ 42.9°C (90.0°F-109.9°F)
Accuracy:	±0.1°C (±0.2°F) during 35.5°C-42.0°C (95.9°F-107.6°F) at 18°C-28°C (64.4°F-82.4°F) ambient operating range ±0.2°C (±0.4°F) for other measuring and ambient operating range
Operating mode:	Direct Mode
Display:	Liquid crystal display, 3 1/2 digits
Memory:	For storing the last measured value
Battery:	One 1.5V DC button battery (size LR41 or SR41, UCC392)
Battery life:	Approx. 200 hours of continuous operation or 1 year with 3 measurements per day
Dimension:	13.8cm x 2.2cm x 1.2cm (L x W x H)
Weight:	Approx. 12 grams including battery
Expected service life:	Three years
Ambient operating range:	Temperature: 5°C ~ 40°C (41°F ~ 104°F) Relative humidity: 15%-95%RH Atmospheric Pressure: 700hPa - 1060hPa
Storage and transportation condition:	Temperature: -20°C-55°C(-4°F-131°F) Relative humidity: 15%-95%RH Atmospheric Pressure: 700hPa - 1060hPa
Ingress Protection Rating:	IP 27
Classification:	Type BF

FEVERLINE INDICATING TECHNOLOGY

At the completion of each measurement, the triangular arrow on the display will indicate different temperature range.

Triangular arrow explanation:

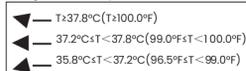For example:

Note: If temperature below 35.8°C (96.4°F), the triangular arrow will not display.

SWITCH UNITS: °C/°F

Temperature readings are available in Celsius or Fahrenheit (°C/°F) located in the upper right corner of LCD.) With the unit off, press and hold the On/Off Button for approximately 2 seconds to change the current setting.

DIRECTIONS

- Press the On/Off Button next to LCD display. A tone will sound as the screen shows 38.8°C, followed by last recorded temperature. After showing the self-test temperature, the thermometer is now in the testing mode.

- Position thermometer in desired location (mouth, rectum, or armpit).
 - Oral Use:** Place thermometer under tongue as indicated by "✓" position shown in Figure 2. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air.
 - Rectal Use:** Lubricate silver probe tip with petroleum jelly for easy insertion. Gently insert sensor approximately 1cm (less than 1/2") into rectum.
 - Armpit Use:** Wipe armpit dry. Place probe in armpit and keep arm pressed firmly at side. From a medical viewpoint, this method will always provide inaccurate readings, and should not be used if precise measurements are required.
- The degree sign flashes throughout the testing process. When flashing stops an alarm will beep for approximately 10 seconds. The measured reading will appear on the LCD simultaneously. The minimum measurement time until the signaling tone (beep) must be maintained without exception. The measurement continues even after the buzzer notification. So that in order to achieve better body temperature measurement result, recommend to keep the probe in mouth and rectum about 2 minutes, or in armpit about 5 minutes regardless of the beep sound and at least 30 seconds measurement interval should be maintained.

*Note: Normally the buzzes are "Bi-Bi-Bi-Bi-"; Alarm beeps more rapidly when temperature reaches 37.8°C (100°F) or higher, and the buzzes are "Bi-Bi-Bi----- Bi-Bi-Bi----- Bi-Bi-Bi".

To prolong battery life, press the On/Off Button to turn unit off after testing is complete. If no action is taken, the unit will automatically shut off after around 10 minutes.

TROUBLESHOOTING

Error message	Problem	Solution
Lo	Temperature taken is lower than 32.0°C (90.0°F)	Turn off, wait one minute and take a new temperature via close contact and sufficient rest.
Hi	Temperature taken is higher than 42.9°C (109.9°F)	Turn off, wait one minute and take a new temperature via close contact and sufficient rest.
Err	The system is not functioning properly.	Unload the battery, wait for 1 minute and repower it, if the message reappears, contact the retailer for service.
	Dead battery: Battery icon is flashing.	Replace the battery.

BATTERY REPLACEMENT

- Replace battery when appears in the lower right corner of LCD display.
- Pull battery cover off as shown in Figure 3.
- Gently pull out plastic circuit board with battery chamber approximately 1 cm (slightly less than 1/2"). (See Figure 4)
- Use a pointy object, such as a pen to remove the old battery. Discard the battery lawfully. Replace with a new 1.5V DC button type LR41 or SR41, UCC392, or equivalent. Make sure the battery is installed with "+" polarity facing up. (See Figure 5)
- Slide the battery chamber back into place, and attach cover.

CLEANING AND DISINFECTION

- Immerse the thermometer probe in distilled water for at least 1 minute;
- Using a clean, soft cloth to wipe the thermometer down to remove any residue;
- Repeat step 1 and 2 for three times until no soil is seen with visual inspection after cleaning;
- For thoroughly clean and disinfection, please use method A or B:
 - Method A (High level disinfection): immerse the thermometer probe in 0.55% OPA (O-Phthaldehyde), such as CIDEX OPA, for at least 12 minutes under temperature at 20°C;
 - Method B (Low level disinfection): Using a clean soft cloth dipped in 70% medical alcohol, wipe the probe 3 times at least one minute for each time.
- Repeat step 1 to 3 to remove OPA residuals;

Note1: Rectal use is not recommended for home use as OPA will not be readily available outside of a hospital. If rectal measurement is necessary, we strongly recommend high level disinfection. Note2: Please operate according to the manual of OPA for reference.

To prevent damage to the thermometer please note and observe the following:
- Do not use benzene, paint thinner, gasoline or other strong solvents to clean the thermometer.
- Do not attempt to disinfect the sensing probe (tip) of the thermometer by immersing in alcohol, OPA or in hot water (water over 122°F (50°C) for long time).
- Do not use ultrasonic washing to clean the thermometer.

LIMITED WARRANTY

The thermometer is guaranteed for one year from the date of purchase. If the thermometer does not function properly due to defective components or poor workmanship, we will repair or replace it free of charge. All components are covered by this warranty excluding the battery. The warranty does not cover damages to your thermometer due to improper handling. To obtain warranty services, an original or copy of the sales receipt from the original retailer is required.

Questions? E-mail support@byfloproducts.com

Disposal of this product and used batteries should be carried out in accordance with the national regulations for the disposal of electronic products.

ByFlo Products Inc. 630- 1200 Rue Saint-Alexandre, Montreal, QC, H3B 3H5 CANADA

Made in China

Document No. BFP-DMT-4132
Version: A
Date of Issue: 2023.05

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